


Directions for Use (Single Patient Use Only)

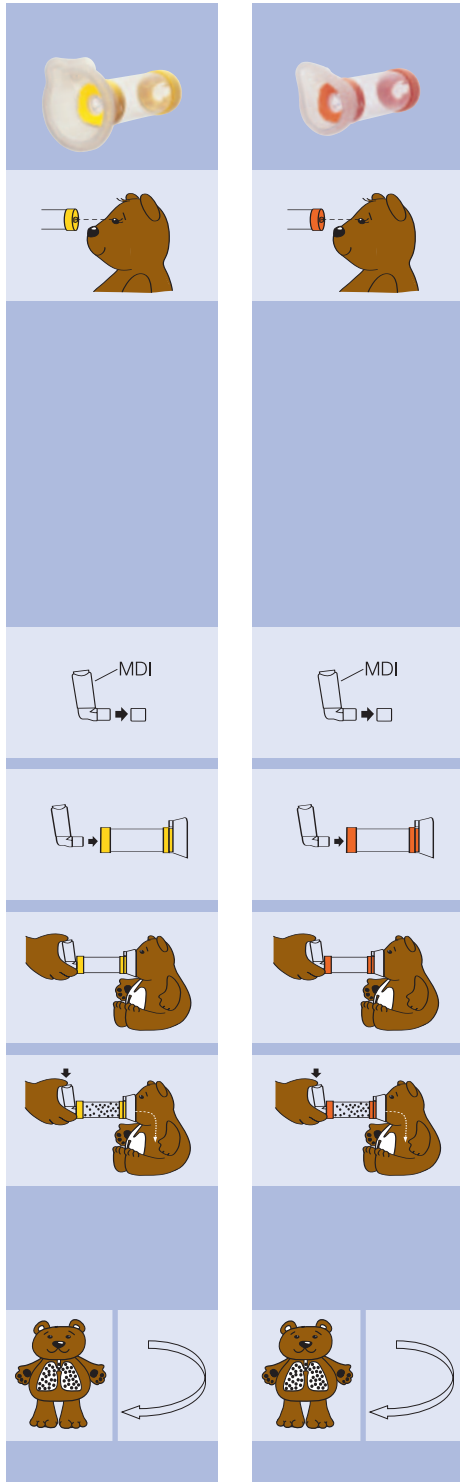

ENGLISH

Carefully examine the product for damage, missing parts or foreign objects. Any foreign objects should be removed. The product should be replaced IMMEDIATELY if there are any damaged or missing parts. If necessary, use the inhaler (MDI) alone until a replacement is obtained. If the patient's symptoms worsen, please seek immediate medical attention.


Before use, make sure that instructions supplied with inhaler (MDI) have been read. Remove cap.

Shake the inhaler (MDI) well immediately before each use. Insert the inhaler (MDI) into the back piece of the chamber.

Apply mask to face and ensure that there is a good seal.

Depress inhaler (MDI) at beginning of slow inhalation. Maintain seal for six (6) breaths after depressing inhaler (MDI). Administer one (1) puff at a time.

Follow instructions supplied with the inhaler (MDI) on amount of time to wait before repeating steps 3–5 as prescribed.

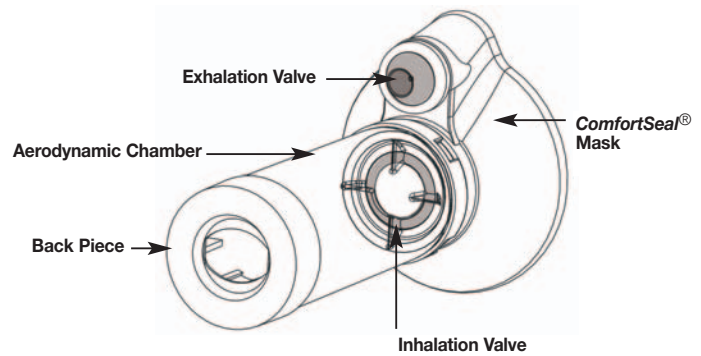

Cleaning Instructions



ENGLISH

Clean **AeroChamber Plus**[®] VHC only as per instructions before first use, then weekly.



Remove back piece only. Do not remove mask or tamper with valves during cleaning.




Soak both parts for 15 minutes in lukewarm water with liquid detergent. Agitate gently.




Rinse in clean water.



Shake out excess water. **Do not rub dry.**

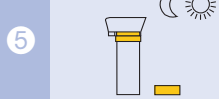

Let air dry in vertical position.

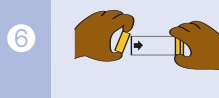

Replace back piece when unit is completely dry and ready for use.



Valved Holding Chamber with **ComfortSeal**[®] Mask



monaghan[™]

Monaghan Medical Corporation, 5 Latour Ave., Suite 1600, Plattsburgh, NY 12901

www.monaghanmed.com

© 1999, 2002 Monaghan Medical Corporation

[™] trademarks and © registered trademarks

Patents 5,645,049; 5,848,588; 6,345,617; 6,293,279; and Patents Pending

Part Number: 76070 AWC33 Revision: 04/04

Cautions

- Read all instructions before use.
- Do not leave **AeroChamber Plus**[®] VHC unattended with children. This is a medical device, not a toy.
- Do not disassemble the product beyond what is recommended in the Cleaning Instructions or damage may result.
- If unsure on how to use this product, talk to your physician or pharmacist.

Notes

- VHC = Valved Holding Chamber
- This product contains no latex.

Indications for Use: The **AeroChamber Plus**[®] Valved Holding Chamber ("VHC") is intended to be used by patients who are under the care or treatment of a licensed health care professional or physician. The device is intended to be used by these patients to administer aerosolized medication from most pressurized Metered Dose Inhalers, prescribed by a physician or health care professional. The intended environments for use include the home, hospitals and clinics. **Rx Only**